

MONTHLY E-MAGAZINE

HYA[•]PROF

BioScience GmbH introduces Hyaprof
a revolutionary range of monophasic dermal fillers

During this year's edition of IMCAS World Congress in Paris, BioScience GmbH introduced their new revolutionary line of monophasic dermal fillers - Hyaprof. The official launch event took place in Pavillon Wagram in Paris and gathered a host of top aesthetic medicine professionals from around the world.



The newest addition to their product portfolio makes BioScience a manufacturer offering both monophasic and biphasic products for the full spectrum of facial and body applications.

About Hyaprof

Hyaprof products combine monodensified and polydensified properties to offer a solution for achieving subtle yet long-lasting results and can be injected with 30G and 27G needles which in turn ensures a painless treatment.

Thanks to the Advanced Thixotropic Technology and Smart Cross-Linking Technology, Hyaprof range demonstrates unique and optimized rheological properties for the improved performance of hyaluronic acid, smooth application as well as seamless dermal integration.

Products

The line consists of two products, which can be used individually or in combination in both superficial dermis and deeper injections, depending on the patient's facial structure and functional needs:

- **Hyaprof Soft** (pure hyaluronic acid: 20mg/mL): a monodensified dermal filler suitable for superficial and submucosal indications. Ideal for the correction of fine lines, crow's feet, perioral lines, fine forehead lines, thin lips, facial atrophic scars, residual rhytids after botulinum toxin as well as fine lines of the lateral periocular and infraorbital regions.
- **Hyaprof Balance** (pure hyaluronic acid: 22mg/mL): a polydensified dermal filler with a greater degree of cross-linking, optimal for injections ranging from mid to deep dermis. Ideal for the correction of moderate and deeper lines, tear through deformity, nasolabial wrinkles, glabellar line, lip contouring, philtrum, oral commissures, marionette folds as well as restoration of facial volume and shape.

Hyaprof dermal fillers line is also accompanied by a skincare system range for post-procedure aftercare.

Bio | SCIENCE

www.biosciencegmbh.com
www.hyaprof.com

The background is a solid magenta color. There are three circles: a small gold circle in the top right corner, a large white circle in the top right quadrant, and a large gold circle in the bottom left corner containing a white circle.

Aesthetics

BioScience launches new range of monophasic dermal fillers

26 FEB 2020



Aesthetic product manufacturer BioScience GmbH has launched Hyaprof, a new injectable line which combines monodensified and polydensified dermal fillers.

The range consists of two products, the Hyaprof Soft and the Hyaprof balance, which the company states can be used individually or in combination for superficial dermis procedures and deeper injections.

The Hyaprof Soft is a monodensified dermal filler suitable for superficial and submucosal indications, designed to correct fine lines, crow's feet, perioral lines, fine forehead lines, thin lips, facial atrophic scars and residual rhytids.

The Hyaprof Balance is polydensified dermal filler with a greater degree of cross-linking. According to BioScience it is suitable for the correction of moderate and deeper lines, tear trough deformity, nasolabial wrinkles, glabellar lines, lip contouring, philtrum treatment, oral commissures and marionette folds.

The company states that the products are designed to improve cohesivity, viscoelasticity and plasticity to allow homogeneous soft tissue augmentation and rejuvenation.

The UK distributors for BioScience products are Rosmetics and DermaMed Solutions.



Guided Solutions



BioScience launches new range of monophasic dermal fillers

By **Guided Solutions Medical Aesthetics** February 27, 2020



bioscience-dermal-fillers

Aesthetic product manufacturer BioScience GmbH has launched Hyaprof, a new injectable line which combines monodensified and polydensified dermal fillers.

The range consists of two products, the Hyaprof Soft and the Hyaprof balance, which the company states can be used individually or in combination for superficial dermis procedures and deeper injections.

The Hyaprof Soft is a monodensified dermal filler suitable for superficial and submucosal indications, designed to correct fine lines, crow's feet, perioral lines, fine forehead lines, thin lips, facial atrophic scars and residual rhytids.

The Hyaprof Balance is polydensified dermal filler with a greater degree of cross-linking. According to BioScience it is suitable for the correction of moderate and deeper lines, tear trough deformity, nasolabial wrinkles, glabellar lines, lip contouring, philtrum treatment, oral commissures and marionette folds.

The company states that the products are designed to improve cohesivity, viscoelasticity and plasticity to allow homogeneous soft tissue augmentation and rejuvenation.

The UK distributors for BioScience products are Rosmetics and DermaMed Solutions.

Tailored for your lips!

Durable • Natural • Pain free



The only thixotropic monophasic dermal filler
Unique HA rheological properties
The only dual specialty dermal filler

The background is a solid magenta color. It features several geometric shapes: a large white circle in the top right corner containing a smaller magenta circle; a small magenta circle in the top right; a small brown circle in the middle right; a small white circle in the bottom left; and a large brown circle in the bottom left corner.

The PMFA Journal

Lip augmentation with hyaluronic acid

Marta Duarte takes us through her technique for lip augmentation, from patient selection to aftercare.

Lips are considered extremely important for maintaining the overall facial aesthetics. It is no surprise that today, lip augmentation procedures have become the most sought-after aesthetic treatment. Soft tissue augmentation using hyaluronic acid (HA) dermal fillers has a remarkable ability to undo the consequences of ageing, such as photodamage, smoking, as well as peripheral rhytides, not to mention increasing the patient's self-confidence. Genetically thin lips, cosmetic asymmetries, and prominent 'marionette lines' can also be corrected non-surgically to help patients achieve results that meet their individual needs and anatomically adhere to ideal proportions.

Lip augmentation using hyaluronic acid dermal fillers is a non-surgical, non-invasive procedure that takes 15 minutes on average. The results are immediate, with minimal side-effects or adverse reactions and efficient outcomes.

Patient selection

While experience and knowledge of a product should result in a successful treatment, the outcome may be unexpected with an unsuitable patient. Careful patient selection is an absolute must. A detailed consultation where benefits, limitations and potential unfavourable results are summarised is necessary pre-treatment. I make sure that I counsel my patients, discussing the immediate aftermath of lip augmentation. I also encourage them to disclose their full medical history of any surgical or non-surgical procedures, fears or concerns, in order to ensure the entire experience is hassle-free and results are as per expectations.

Besides the concerns outlined above, the ideal candidate for lip filling is one whose lip structure is inherently thin or patients whose volume has diminished with age. It is essential that the patient does not have any facial dermal infections, uncontrolled blood clotting disorders, uncontrolled diabetes, HA sensitivity, and has not taken blood-thinning medication (ibuprofen or aspirin) up to two weeks prior to the procedure.

Preparation

It is crucial to understand the patient's needs. Some patients could be looking for subtle corrections while others for enhanced definition or fullness. I always make sure to identify the exact requirement and advise the best course of treatment in advance.

Before pictures are taken, previous procedures are discussed and the respective consent form, with all the possible side-effects listed, is signed by the patient.

Lip augmentation is performed under local anaesthesia or by applying anaesthetic creams, depending on the patient's pain tolerance. My personal preference is to use dental block with infraorbital and mental nerve block, as frequently with anaesthetic creams patients are not comfortable enough and this may result in an inaccurate treatment.

Procedure (technique of injection)

For optimal lip augmentation results, it is necessary to maintain proper contour and balance between the upper and lower lips. The golden ratio for female lips is the ratio of 1.0 upper lip for 1.6 of lower lip (for most ethnicities). It is imperative to correct lips while maintaining the natural ratio and proportion. In my experience, a deflating vermilion is one of the biggest patient concerns, comprising of drooping angles of the mouth. More often than not, it is the upper lip that becomes the focus of the treatments rather than the lower lip.



Patient before and after treatment.

The technique for injecting HA fillers into the lips involves serial punctures and linear threading, preferably retrograde. These can be applied either individually or in combination based on the patient's needs.

Upon assessing the lips, it can be ascertained if the lips require superficial or deep filling. The best approach is an individual approach, paying attention to each movement of the needle and keeping in mind the patient's aesthetic desires. For especially thin lips, volume augmentation with deeper-placed filler is advised, followed by surface corrections to add definition. Contrarily, for enhancement purposes, a superficially placed filler with emphasis on the vermilion border is better.

I prefer to use a 27G cannula instead of a needle as its blunt tip minimises the trauma and reduces any risk of penetrating through the vessels. It also allows access to each area of the lip while minimising the need for skin penetration. In most cases, only one point is required on each side of the lip. In my opinion, filler should not be injected from the border of the lips unless an elongated appearance to the lips is required or in the case of attempting to provide structural support to the augmented lips. In the end, I ensure that the lips are evenly augmented by requesting that the patient smiles and puckers their mouth, which can confirm that the curvature is even and the proportions are maintained.

In my experience, I have been able to obtain optimal results with Hyapro Balance, a polydensified monophasic dermal filler that gives a natural look and ensures long-lasting results.

Aftercare

Immediately after treatment, swelling or bruising may appear even when using the correct technique. Ice-packs can be applied to the area to minimise any discomfort. You may recommend the patient avoids additional treatment, lip exfoliation and extreme temperatures within 24 hours of the treatment.

AUTHOR



Marta Sofia Fernandes Duarte,

Medical Doctor, Certified Aesthetic Physician and Certified Laser Specialist; Founder, Dr Marta Beauty – Aesthetic Clinic; Fellow in Aesthetic Medicine and Advanced Injectables, American Academy of Anti-Aging Medicine; Certified in Aesthetic Gynecology, New York Aesthetic Rejuvenation Institute; Currently working Dubai, UAE and Lisbon, Portugal.

Declaration of competing interests: None declared.



ISAPS News

ISAPS[®]NEWS

OFFICIAL NEWS OF THE INTERNATIONAL
SOCIETY OF AESTHETIC PLASTIC SURGERY

4

Volume 14 | Number 4



INSIDE ■

Global Perspectives:
Rhinoplasty

Spotlight on
National Secretaries

History of Rhinoplasty

New Directions
in Education

HYAPROF

THE COUTURE LIPS



Hyaprof is a revolutionary couture lip-filler that is designed to provide a tailored solution to every individual's specific proportions and requirements for the lips. The product line is supported by the patented technologies of ATT and SCLT that help ensure the longest-lasting results.

With the benefit of dual specialty, Hyaprof is one stop-solution for the complete lip enhancement treatment —Hyaprof Soft contours the lips for a perfectly defined appearance while Hyaprof Balance augments and plumps them for the ideal fuller-lip look.